



# BATTERY REPLACEMENT

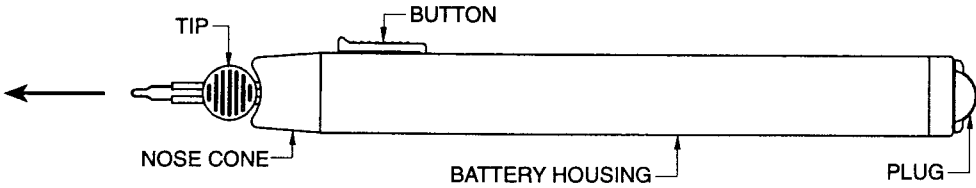


1. REMOVE TIP before beginning battery replacement procedure.
2. To remove depleted batteries, simply push in and turn the rear plug left 1/4" (counter clockwise).

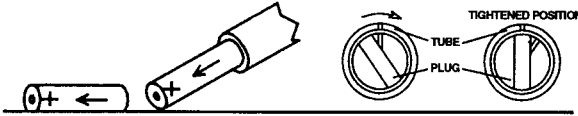


3. Pull out the plug and tilt the power handle so the batteries slide out. A gentle downward shake may be required in some instances.

4. Insert new AA ALKALINE BATTERIES negative end first (never use carbon zinc batteries in this product).

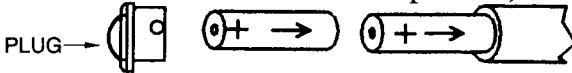

5. Align plug with tabs and insert, then turn right 1/4" (clockwise). DO NOT FORCE, there is only one correct way, and it should be easily located.

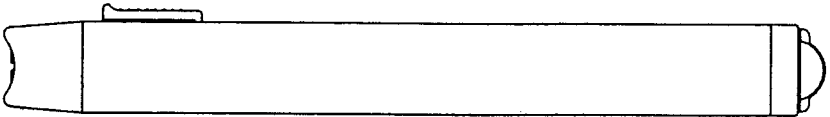

**YOUR POWER HANDLE IS NOW READY FOR USE.**

**IMPORTANT: ALWAYS REMOVE TIP  
WHEN IN STORAGE OR NOT IN USE.**

**\*\*WARNING: DO NOT USE IN THE PRESENCE OF  
FLAMMABLE MATERIALS - DO NOT AUTOCLAVE**