




# TIP REPLACEMENT

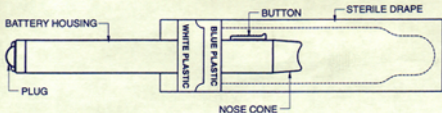

1. Remove the Sterile Drape from the peel pouch.
2. Holding the Drape with the blue stripe up, begin sliding the power handle BETWEEN THE WHITE PLASTIC AND WHITE PAPER TABS.
3. GENTLY slide the power handle forward, with a back and forth rotation, until the nose of the power handle is at the tip of the Sterile Drape.
4. Peel away the blue stripe section. Peel away the white paper backing.
5. Be careful handling tips, as the wire is fragile. ALWAYS HOLD THE TIP BY THE PLASTIC PART TO AVOID DAMAGE.

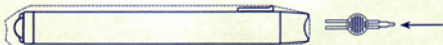

6. Insert the Sterile Cautery Tip through the Sterile Drape. DO NOT FORCE, it should easily slide into place.
7. Unit is activated by depressing white button.

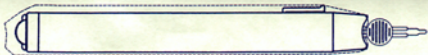

**YOUR CAUTERY UNIT IS NOW READY FOR USE.**

**IMPORTANT: ALWAYS REMOVE TIP  
WHEN IN STORAGE OR NOT IN USE.**

**\*\*WARNING: DO NOT USE IN THE PRESENCE OF  
FLAMMABLE MATERIALS - DO NOT AUTOCLAVE**